

ClearPath System Nx-301 Sensor Instructions

NexiSensor Application


You should read these instructions before applying the sensor.

WARNING: DO NOT APPLY SENSOR TO BROKEN, DAMAGED OR IRRITATED SKIN. REMOVE SENSOR IF SKIN IRRITATION OCCURS.

CAUTION: Do not use lotions, oils, creams, perfumes, deodorant or powder on the area where the Sensor is being applied.

Step 1. Check contents of bag:

BAG - Contains a Sensor (mounted on a thin plastic Application Aid), a small Sticky Velcro Patch & packets of alcohol prep wipes.

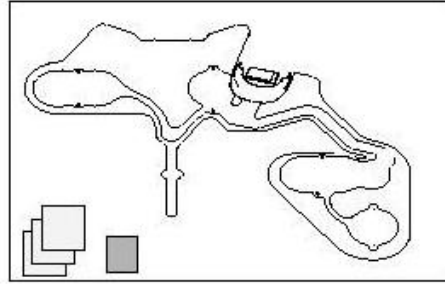


Step 2. Prepare Skin for Sensor:

CAUTION: Wait 40 minutes after a bath or shower before applying a Sensor.

SHAVE – If hair is present shave hair from **Skin Contact Sites**.

CLEAN - Use unfolded supplied wipe to wipe **Skin Sites** clean.


RUB DRY- Rub briskly with a clean towel or cloth until **Skin Sites** are slightly red.

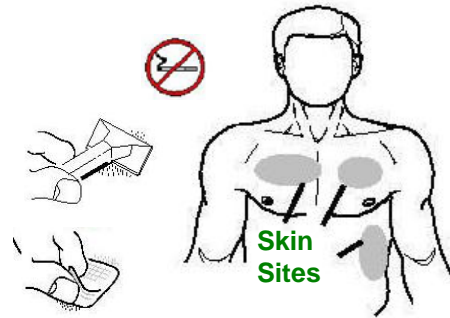

Step 3. Ensure NexiSensor/Sender are working correctly:

PATCH - Place the Sticky Patch (fabric side down) onto the Velcro® on the back of Sender above the label.

SENSOR - Lay it on a flat surface, Clip the Sender to the Sensor. (see diagram opposite ⇒)

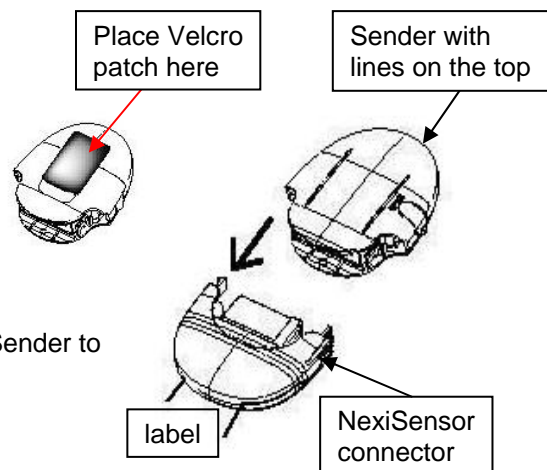

ClearPath System Nx-301 Sensor Instructions


Step 4. Check system working (requires a Partner):

Using the green button turn the Partner 'on'. With the Partner switched on, a 'smiley face' should appear on the screen about 10 seconds after the NexiSensor and Sender are connected. If no Smiley Face appears see [troubleshooting](#) at the end of this document



Step 5. Check Position

APPLICATION AID – Stand in front of a mirror, and place the sensor on your chest with the application aid outermost, use the mirror to check the position of the Sensor ensure the notches at the top of the Aid, line up with the bottom of your two collarbones (see ① on Aid). Ensure the 'Cross' is in middle of chest.

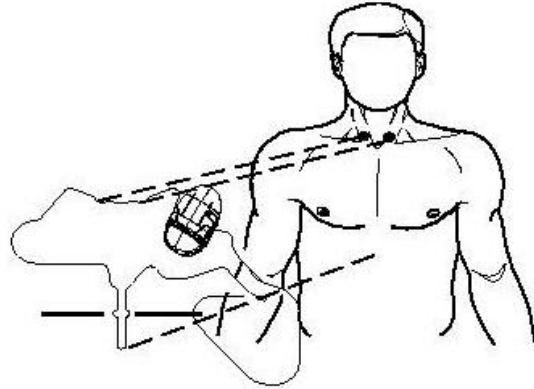


Step 6. Applying the Sensor

Hold the sensor against your chest

First: Peel off the paper strips from chest parts of the Sensor and stick the electrode patches to your chest one at a time.

Second: Peel off the backing strips from the Sender Velcro patch and stick it to your chest. Firmly press and smooth the Aid across chest.

Thirdly: Remove underarm paper strip. Firmly press and smooth the Aid across the body and ensure the ② on the 'Cross' is level with the ② on the underarm part.

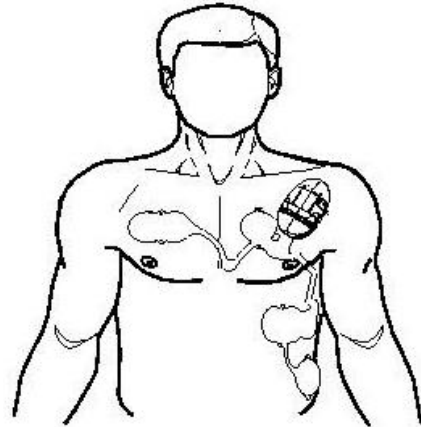

Your Sensor is now attached and working.