
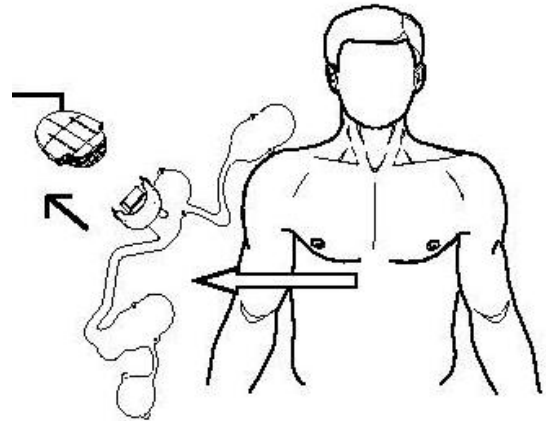


ClearPath System Nx-301 Sensor Instructions

Removal of the NexiSensor and Sender

Removal after 24 hrs:

- Gently peel off the Sender and Sensor from body.
- Unclip the Sender from the NexiSensor by pressing the two side-clips (see red arrows) and pulling the two halves apart.
- Retain or Return the Sender (and Partner) as directed by your physician.
- Shower or wash areas where the NexiSensor and Sender were attached.
- The NexiSensor assembly contains alkaline batteries and should be disposed of in accordance with local regulations.

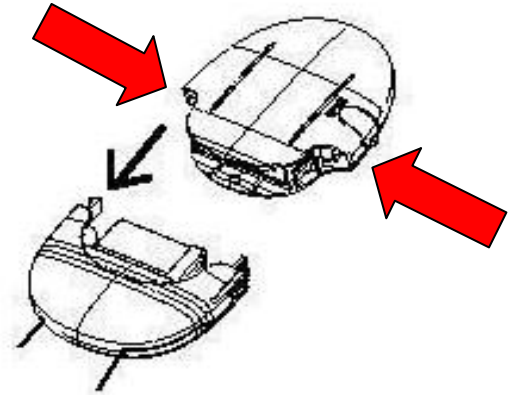

For further details see the [PATIENT GUIDE](#) for the ClearPath System Nx-301



Do not re-use Sensor, for single use only.